
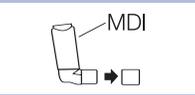
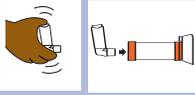
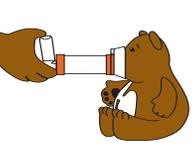
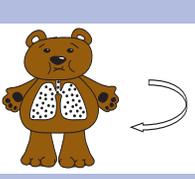




Directions for Use (Single Patient Use Only)

Product Reorder Numbers
Small VHC- PN 88710
Medium VHC- PN 78710

ENGLISH	
	<p>Important: Follow instructions that came with inhaler (MDI).</p> <p>Carefully examine the product for damage, missing parts or foreign objects. The product should be replaced IMMEDIATELY if there are any damaged or missing parts. If necessary, use the inhaler (MDI) alone until a replacement is obtained. If symptoms worsen, please seek immediate medical attention.</p>
<p>1</p> 	<p><i>Before use, ensure the instructions supplied with the MDI have been read.</i> Remove cap from the MDI.</p>
<p>2</p> 	<p>Shake the MDI immediately before each use as per the instructions supplied with MDI. Insert the MDI into the backpiece of the chamber.</p>
<p>3</p> 	<p>Apply mask to face and ensure there is an effective mask/face seal.</p> <p>While maintaining an effective mask/face seal, the patient should exhale through the mask.</p>
<p>4</p> 	<p>Depress inhaler (MDI) at the beginning of slow deep inhalation. Maintain the effective mask/face seal and ensure an additional 5-6 breaths are taken through the chamber before administering another puff of medication.</p> <p><i>Administer one (1) puff at a time.</i></p>
<p>5</p> 	<p>Follow instructions supplied with the inhaler (MDI) on amount of time to wait before repeating steps 3-6 as prescribed.</p>
<p>6</p> 	

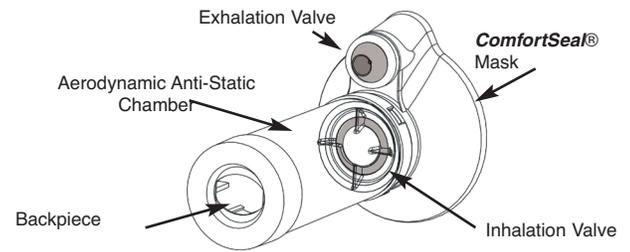

Cleaning Instructions

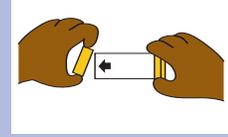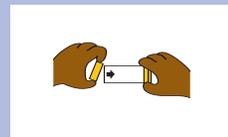ENGLISH	
	<p>The AeroChamber Plus® Z STAT™ aVHC should be cleaned weekly, following the instructions provided below.</p>
<p>1</p> 	<p>Remove the flexible backpiece only. Do not remove the mask or tamper with valves during cleaning.</p>
<p>2</p> 	<p>Soak both parts for 15 minutes in lukewarm water with liquid detergent. Agitate gently.</p>
<p>3</p> 	<p>Rinse parts in clean water.</p>
<p>4</p> 	<p>Shake out excess water. Do not rub dry.</p>
<p>5</p> 	<p>Let air dry in vertical position.</p>
<p>6</p> 	<p>Replace back piece when unit is completely dry and ready for use.</p>

AeroChamber Plus® Z STAT™ 

anti-static Valved Holding Chamber with Small or Medium **ComfortSeal®** Mask



monaghan™

Monaghan Medical Corporation
5 Latour Ave., Suite 1600
Plattsburgh, NY 12901
For additional information visit www.monaghanmed.com

© 2005 Monaghan Medical Corporation
™ trademarks and ® registered trademarks
Patent Numbers 5,645,049; 5,848,588; 5,988,160; 6,293,279;
6,345,617; 6,435,177; and Patents Pending

Printed in USA Part Number: 87703-00 AWA484-00 03/07

Cautions

- Read all instructions before use.
- Product should be replaced if valves are damaged or missing.
- Do not leave the **AeroChamber Plus® Z STAT™** aVHC unattended with children.
- If unsure on how to use this product, talk to your physician, pharmacist or health care provider. This is a medical device, not a toy.
- Do not disassemble the product beyond what is recommended for cleaning, or damage may result.
- **Federal (USA) law restricts the sale of this device on or by the order of a Physician.**

Notes

- aVHC = anti-static Valved Holding Chamber
- Storage and operating range 5°C - 40°C (41° F - 104°F) at 15 to 95% relative humidity.
- **THIS PRODUCT CONTAINS NO LATEX.**
- Product may need to be replaced after 12 months of continuous use. Environmental conditions, storage and proper cleaning can affect product life span.

Indications for Use: The **AeroChamber Plus® Z STAT™** anti-static Valved Holding Chamber ("aVHC") with **ComfortSeal®** Mask is intended to be used by patients who are under the care or treatment of a licensed health care professional or physician. The device is intended to be used by these patients to administer aerosolized medication from most pressurized Metered Dose Inhalers, prescribed by a physician or health care professional. The intended environments for use include the home, hospitals and clinics.

Rx ONLY

Product Performance Data is available upon request