

# 3M™ Tegaderm™ Superabsorber Superabsorber Dressing

## GENERAL:

Tegaderm™ Superabsorber is a sterile Superabsorber Dressing for use on moderate to heavily exuding wounds.

## INDICATIONS:

Exuding wounds such as leg ulcers, pressure ulcers, diabetic foot ulcers and other wounds producing high levels of exudate.

## PRODUCT PROPERTIES:

Tegaderm™ Superabsorber absorbs and retains exudate in the dressing. The product wicks exudate vertically from the white wound contact side, retaining it and forming a gel within the dressing. This minimizes the risk of maceration and damage to periwound skin and provides a moist environment.

## INSTRUCTIONS FOR USE:

Tegaderm™ Superabsorber should be used on moderate to heavily exuding wounds when there is a need for high absorption and fewer dressing changes. Tegaderm™ Superabsorber can be used both – as a primary dressing or as a secondary dressing.

Observe: Tegaderm™ Superabsorber has a white wound contact side and a hydrophobic green backing (side up). The backing minimizes moisture leakage. Choose a size of dressing depending on the wound size; the core edge of the dressing should overlap the wound by minimum 2cm (0,8").

Tegaderm™ Superabsorber must be in close contact with the wound bed or the primary dressing. Secure the product in place. Tegaderm™ Superabsorber may be used in combination with compression therapy.

The dressing should be monitored regularly in accordance with best practice in the management of exuding wounds to avoid complete saturation. Dressing removal can be facilitated by moistening the dressing with saline solution. Dispose of as normal waste. If contaminated, follow local regulations.

## CAUTION:

Important to use the white wound contact side towards the wound! Do not cut the dressing, the cover must remain intact. Do not use the dressing if the sterile packaging is damaged. Single use product, do not re-sterilize, cannot be re-used.

## CONTRAINDICATIONS:

The dressing should not be used on dry or low exuding wounds, eyes, mucous membranes, tendons or in wound cavities because the dressing swells during absorption. The dressing should not be used on persons with a known sensitivity to the dressing or its components.

## EXPLANATION OF SYMBOLS:



Caution, see instructions for use




Do not reuse



Do not use if package is damaged or open



Keep away from sunlight

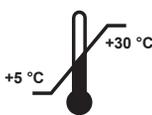

Temperature limitation



Humidity limitation



Manufacturer



Sterilized using ethylene oxide

**REF** Catalog Number

**LOT** Batch code

 Use by

CE 0297

Made in Sweden for

 **3M Deutschland GmbH**  
Health Care Business  
Carl-Schurz-Str. 1  
41453 Neuss  
Germany

3M Health Care  
2510 Conway Ave  
St. Paul, MN 55144  
1-800-339-3957

Issue Date: 2013-09  
DV-2063-0166-0/1013