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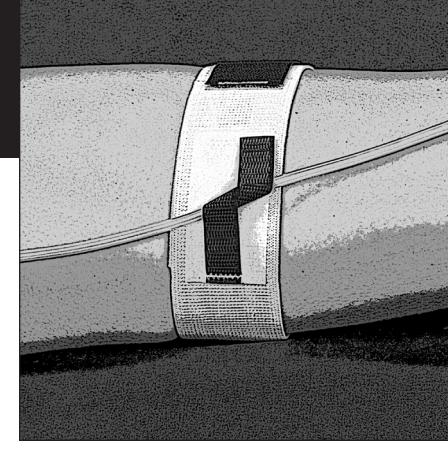
Manufacturer:
DeRoyal Industries, Inc.
200 DeBusk Lane
Powell, TN 37849 USA
888.938.7828 or 865.938.7828
www.deroyal.com

QNET BV
Hommerterweg 286
6436 AM Amstelveen
The Netherlands

Part#0-0721G
Revised 5-2015
©2015 DeRoyal Industries, Inc.
All Rights Reserved.

CE
5-2015

CATHETER STRAP**RX ONLY
SINGLE PATIENT USE
NON-Sterile
NOT MADE WITH NATURAL RUBBER LATEX****IMPORTANT INFORMATION**
Please read all instructions, warnings, and precautions before use. Correct application is essential for proper functioning of the product.

Use only on the person it was provided to by a healthcare professional and only for the use it was intended.

INTENDED USE
The DeRoyal Catheter Strap is intended to be prescribed by a properly licensed practitioner to help secure catheter tubes in place.

PRECAUTIONS
Unnecessary movement of the catheter can cause infection or other discomfort.

This product is to be fitted initially by a physician (or properly licensed practitioner) who is familiar with the purpose for which it is intended. The physician or practitioner is responsible for providing wearing instructions and precautions to other healthcare practitioners or healthcare providers involved in the patient's care as well as the patient. Consult your physician (or properly licensed practitioner) immediately if you experience sensation changes, unusual reactions, swelling or increased pain while using this product. Follow the instructions of your physician (or properly licensed practitioner) for length and duration of use.

Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

INSTRUCTIONS FOR USE

- Measure the circumference of the patient's thigh to determine the appropriate size.
- Position the catheter strap behind the patient's thigh with the wide blue tab on the medial side of the patient's leg.
- Wrap the elastic catheter strap around the thigh and secure with closure.
- Position the catheter vertically atop the sewn end of the blue tab.
- Wrap the tab around the catheter strap and secure the closure.
- Place the small white foam piece on top of the exposed tab that wraps around the catheter strap.
- The catheter should have the ability to move with the patient to help prevent bladder traction.

SIZING

Size	Dimensions
Universal	22" L x 2" W (56 x 5 cm)
Bariatric	38" L x 2" W (100 x 5 cm)

CLEANING AND/OR MAINTENANCE

Prior to washing, remove any detachable rigid stays.

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TAILLE

Taille Dimensions
Universel 22" Long. x 2" Larg. (56 x 5 cm)
Bariatrique 38" Long. x 2" Larg. (100 x 5 cm)

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