

Product  
Number **H84103161**

# Dale<sup>®</sup> Foley Catheter Holder

with Exclusive Green Locking Tabs

## Product Description

Features patented green locking device that securely positions the foley catheter on the patient. Disposable. Available as legband or waistband.

## Application Instructions

1. Position legband high around patient's thigh with Dale product label pointing to inside of leg. Stretch legband in place and fasten Velcro<sup>®</sup> tab. Proper compression has been achieved when two fingers fit snugly under the band.
2. Place foley catheter over white stitching of green tab. Leave an ample loop in catheter above legband to avoid bladder traction; or stretch catheter for traction when required.
3. With catheter in desired position, insert the narrow, green tab over catheter and through the square opening so that green tabs overlap.
4. Lock catheter in position by pulling green tabs in opposite directions. Secure tabs in place. To reposition catheter: raise either green tab, adjust catheter, and refasten tab.
5. Another method of securement is to locate the Y port of the proximal end of the catheter into the locking device.

**Additional Benefits** To help prevent meatal irritation, legband can be rotated in either direction. Alternating legs periodically may also add to greater patient comfort.

**For single patient use.**

## Ordering Information

#H8410**3161** Dale<sup>®</sup> Foley Catheter Holder/Legband

**HCPCS Code: A4334**

One size fits most. Individually packaged.  
Sold in boxes of 10.

#H8410**3301** Dale<sup>®</sup> Foley Catheter Holder/Bariatric Legband  
**Can also be used as waistband**

**HCPCS Code: A4335**

Legband—fits most bariatric. Waistband—one size fits most. Individually packaged. Sold in boxes of 10.

Dale medical products are available from your preferred hospital/medical supply dealer. **Latex Free.**

US Patent Numbers 4,445,894, and 5,941,856.

Velcro<sup>®</sup> is a registered trademark of VELCRO USA, Inc.



**DALE MEDICAL PRODUCTS, INC.**

7 Cross Street, P.O. Box 1556, Plainville, MA 02762-0556 USA

TEL: 800-343-3980 INT'L: 508-695-9316 FAX: 508-695-6587

[www.dalemed.com](http://www.dalemed.com)



**EU Representative:**  
MDSS GmbH, Burckhardtstr. 1  
D-30163 Hannover, Germany.

Tel: ++49-511-6262-8630 FAX: ++49-511-6262-8633

REV G

