

# Safe n Simple™ How to Use...

## Peri-Stoma Cleanser & Adhesive Remover

**After removing the ostomy appliance, clean your skin with the Peri-Stoma Cleanser & Adhesive Remover wipe.**

These large soft wipes remove oil, dirt and adhesive residue from the skin so the ostomy pouch will adhere well to the skin.

### Instructions for use:

1. Peel back the label slowly
2. Remove wipe
3. Reseal package to keep wipes moist
4. Gently wipe skin clean
5. No need to rinse skin
6. Make sure skin is dry before going on to using a skin barrier
7. Used wipe is flushable (if not on a septic system)



Item# SNS00525 - 50 Wipes/Package  
Item# SNS00505 - 5 Wipes/Package

*More than one wipe may be needed, depending on how much adhesive is on the skin. This wipe may also be used to help remove the appliance from the skin. This is done by slowly starting to gently lift off the appliance/flange and using the wipe between it and the skin to help ease the appliance/flange off.*

## No-Sting Skin Barrier Film

**Leave a protective barrier on the skin to help protect sensitive skin from trauma caused by removal of adhesive from the skin surface.**

Three sizes available with the same formula designed for ease of use.

### Instructions for use:

1. Open package (Large wipe - Peel back the label slowly)
2. Remove wipe
3. Discard sachet packaging or Reseal large wipe package to keep wipes moist
4. Gently wipe/spray skin to apply thin protective barrier
5. Allow skin to dry completely before applying tapes, adhesives or ostomy appliance

*For optimum effectiveness, use Peri-Stoma Cleanser & Adhesive Remover before the No-Sting Barrier Film.*

*If on a septic system, we do not recommend flushing any wipes.*



Item# SNS00807 - 25 Wipes/Package  
Item# SNS80725 - 25 Sachets/Box  
Item# SNS80744 - 100 Sachets/Box  
Item# SNS80792 - 2oz. Bottle

**Safe n Simple™**  
Clean. Comfortable. Confident.

Clarkston, MI 48348 • P: 248.214.4877 • F: 248.698.3454

[www.SnS-Medical.com](http://www.SnS-Medical.com)

*Both types of wipes are covered by insurance, Medicare & Medi-Cal (California's Medicaid Program) which is handled by our distributors. Insurance plan coverage may vary and may not apply to all individuals.*