


TRACHEOTOMY MASK


REF MASK 007A (Adult Mask)
MASK 007P (Pediatric Mask)

Pull the flow restrictor away from the aerosol connector. It may be difficult to pull the restrictor. If the end-user is having difficulty they can do one of the following.

- a. Turn the aerosol connector-restrictor up. This will allow for a straight path to push out the restrictor. Use a clean/sanitized straight object to push out the restrictor.

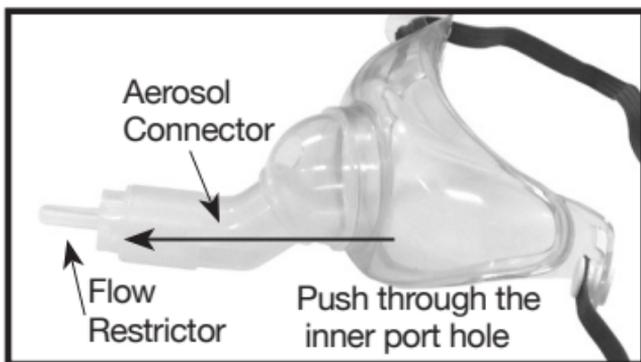

- b. Disconnect the aerosol connector while the restrictor is also attached. Use a finger or a straight object to push out the restrictor.

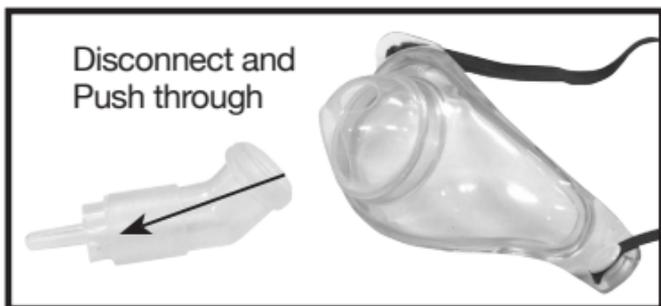

drive[™]

Drive Medical Design & Manufacturing